

QUIKTORK™

Guide Wire Torque Device



One-handed control

The quick *press and release* handling system offers an easy and efficient alternative to the current torque device options.

Two options available:

- The yellow QuikTork™ is used with guidewires with diameters between .010" and .020"
- The orange QuikTork™ is used with guidewires with diameters between .025" and .040"

Instructions for use:

1. Press open end firmly and insert over guidewire.
2. Position torque device and release.
3. Press finger grips to adjust or remove.

Ordering information:

TOR-100-10	Accepts guidewire diameter .010"-.020" Yellow	10/box
TOR-100-25	Accepts guidewire diameter .025"-.040" Orange	10/box

MKT-100-74 Rev Orig.



For more information about QuikTork™ Torque Device or to set up a meeting with a Galt Sales Consultant call (800) 639-2800.